Instructions for Use

FEMALE PYLONS

EN		Instructions for Use	3
DE		Gebrauchsanweisung	6
FR		Notice d'utilisation	9
ES		Instrucciones para el uso	12
IT		Istruzioni per l'uso	15
NO		Bruksanvisning	18
DA		Brugsanvisning	21
SV		Bruksanvisning	24
EL		Οδηγίες Χρήσης	27
FI		Käyttöohjeet	30
NL		Gebruiksaanwijzing	33
PT		Instruções de Utilização	36
PL		Instrukcja użytkowania	39
CS		Návod k použití	42
TR		Kullanım Talimatları	45
RU		Инструкция по использованию	48
JA		取扱説明書	51
ZH		中文说明书	54
KO		사용 설명서	57
AR		تعليمات الاستخدام	60

ENGLISH

PRODUCT DESCRIPTION

The pylons are a range of adapters used as part of a prosthetic system. They are used to connect and align the core elements of the device.

INDICATIONS FOR USE

Lower limb absence.

CONTRAINDICATIONS

None known.

GENERAL SAFETY INSTRUCTIONS

- Read the instructions for use before using the device.
- Only use a pylon for a single patient. The pylons are designed and tested for single patient usage.
- Only use with Össur adapters. The durability of the device cannot be guaranteed if non-Össur adapters are used.
- If any problems occur with the use of this products, immediately consult a medical professional.
- If unusual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist.

WEIGHT LIMITS AND ACTIVITY LEVELS

The pylons can be used for all activity levels.

100 kg/ 220 lb. weight limit: A-742010 Female Pylon AL-Short; A-742020 Female Pylon AL-Long; A-748010 Female Pylon AL/TI-Short; A-748020 Female Pylon AL/TI-Long

166 kg/ 375 lb. weight limit: A-748030 Female Pylon Titanium, WP-Short; A-748040 Female Pylon Titanium, WP-Long

CUTTING AND ASSEMBLY

Shorten the tube to agree with measurements.

CAUTION: Do not damage the tube. If you damage the tube, it can be difficult to put it into the tube clamp adapter.

1. Clamp the tube with a 30 mm cutting jig or into a tube cutter.
2. Make sure to cut the tube straight with a hacksaw or tube cutter.
3. Fully remove all burrs with a file before you insert the pylon into a tube clamp adapter.
4. Make sure the tube is fully inserted and fully touches the base of the tube clamp adapter.
5. Apply Loctite 243 to the set screws. Torque two times to 15 N m to fasten the set screws.

NOTE: If you use bolts for extreme alignment, also torque two times to 15 N m.

SPECIFICATIONS

Item nr.	Item description	Product weight	Product length	Material	Environmental conditions
A-742010	Female Pylon AL-Short	193 g/ 6.81 oz.	250 mm/ 9.84"	Aluminum	Weatherproof
A-742020	Female Pylon AL-Long	268 g/ 9.45 oz.	400 mm/ 15.74"		
A-748010	Female Pylon AL/TI-Short	194 g/ 6.84 oz.	250 mm/ 9.84"	Aluminum, titanium	Waterproof
A-748020	Female Pylon AL/TI-Long	267 g/ 9.42 oz.	400 mm/ 15.74"		
A-748030	Female Pylon Titanium, WP-Short	258 g/ 9.10 oz.	250 mm/ 9.84"		
A-748040	Female Pylon Titanium, WP-Long	374 g/ 13.10 oz.	400 mm/ 15.74"		

ENVIRONMENTAL CONDITIONS

Weatherproof

For pylons: A-742010; A-742020

CAUTION: Do not submerge a weatherproof pylon in water. This can cause corrosion. A corroded pylon can break.

A weatherproof device allows use in a wet and/or humid environment but does not allow submersion. Fresh water splashing against the enclosure from any direction shall have no harmful effect.

Waterproof

For pylons: A-748030; A-748040; A-748010; A-748020

A waterproof device allows use in a wet and/or humid environment and temporary (<30 mins.) submersion of up to 1 m in fresh water. Exposure to fresh water under these circumstances shall have no harmful effect.

All Product:

The pylons should be dried thoroughly after contact with fresh water.

Fresh water: Includes tap water. Excludes salt and chlorinated water.

It is recommended to rinse the product with fresh water and then dry thoroughly after use in wet and/or humid environment

COMPLIANCE

Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies.

Compatibility and compliance with these standards are only achieved if Össur products are used with other recommended Össur components.

These components have been tested according to ISO10328 standard to 3 million load cycles. Depending on the amputee's activity this corresponds to a duration of use 3–5 years. We recommend carrying out regular yearly safety checks.

ISO 10328 - “P” - “m”kg *)

*) Body mass limit not to be exceeded!

For specific conditions and limitations of use
see manufacturer's written instructions on
intended use!

ISO 10328 – label

Item nr.	Weight (kg)	Label text
A-742010	100	ISO 10328-P5-100-kg
A-742020		
A-748010		
A-748020		
A-748030	166	ISO 10328-P7-166-kg
A-748040		

DISPOSAL

All components of the product and packaging should be disposed of in accordance with respective national environmental regulations. Users should contact their local governmental office for information on how these items can be disposed of in an environmentally sound manner.

WARRANTY

2-year limited warranty.

LIABILITY

The manufacturer recommends using the device only under the conditions specified and for the intended purposes. The device must be maintained according to the instructions for use. The manufacturer is not liable for damage caused by component combinations that were not authorized by the manufacturer.

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EN – Caution: Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If unusual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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