
Parts Included



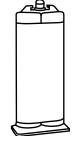

Coyote Quik Glue 50cc

Tech Tip -

Store all glue in cool environment for longevity.

Manufactured by



419 N. Curtis Rd., Boise, Idaho 83706 (208) 429-0026 | www.coyotedesign.com



QUIK GLUE CD4250

BLACK - 50cc - 60 second set time



Made in U.S.A. External Prosthetic Components



CD4250.revD.01142020




Advena Limited Tower Business Centre 2nd Flr, Tower Street Swatar, BKR 4013 Malta


Instructions



Purge the cartridge Snip off the tip. Insert cartridge into dispenser gun. Expel a small amount of adhesive to purge the cartridge.

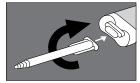

Install mix tip
Line up mix tip appropriately
and twist into place.
Tips can only be used once.

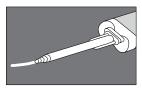

Run a test bead approx. 1" in length. Store unused portion by leaving the mix tip on when done



Leave used tip attached as a cap until next use to keep glue from drying out in the tubes.

When your ready to use the glue again remove old tip and replace with a new tip.




CAUTION

- 1. Read carefully BEFORE applying adhesive to ensure proper adhesion.
- 2. Quik Glue sets fast. To ensure a proper bond, work quickly. Refrigerate the cartridge to slow down work time.

Parts Sold Separately

CDDMA51 | Dispenser is required for use (sold separately)

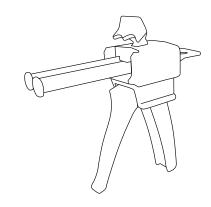

CD401 | 6-Pack Extra Mix Tips for 50 cc Cartridge.

CD402 | 12-Pack Extra Mix Tips for 50 cc Cartridge.



Contact Coyote for a MSDS for this product. (208) 429-0026 or mailbox@coyotedesign.com