


To determine proper liner sizing, two measurements are required:

Step #1: Take circumference measurement 2" (5-6cm) proximal to the distal end of the residual limb.

Step #2: Take length measurement from mid-patella tendon to the distal end of the residual limb.

Note: All Soflex liners come with full fabric cover, in either locking or cushion (non-locking) versions.

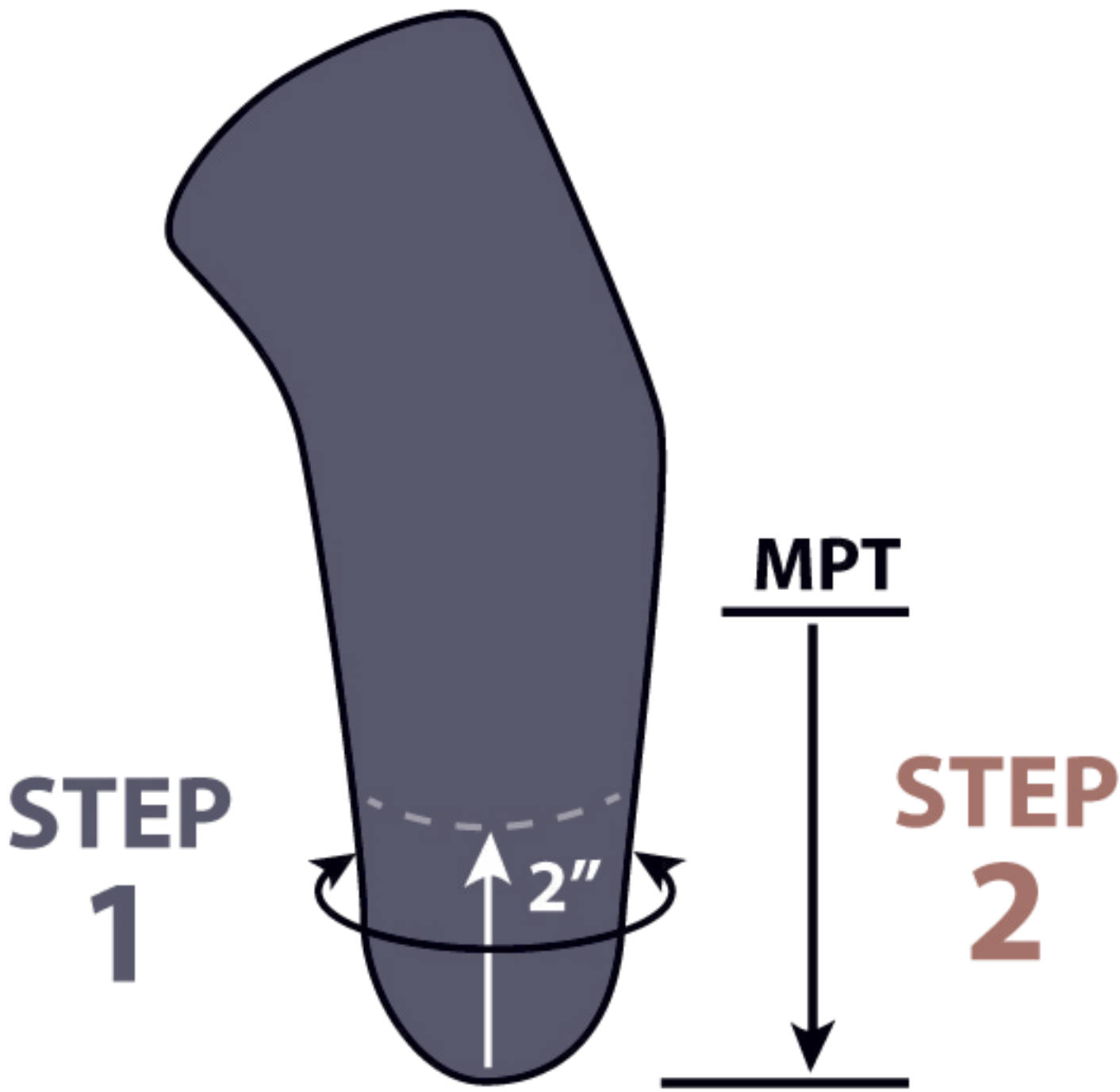

EXAMPLE: SBK20 - 6 L

Limb Circ. Of 24" and an MPT to Distal Length of 6" plus Textile with Lock.

STEP 1: Limb Circ.	Choose Part Number
17 - 19 cm	SBK14 - A B
19 - 21 cm	SBK16 - A B
21 - 23 cm	SBK18 - A B
23 - 25 cm	SBK20 - A B
25 - 27 cm	SBK22 - A B
27 - 29 cm	SBK24 - A B
30 - 32 cm	SBK26 - A B
32 - 34 cm	SBK28 - A B
35 - 37 cm	SBK30 - A B

STEP 2: MPT to Distal	Insert # for "A"
4"	4
6"	6
8"	8
10"	10*

*SoFlex sizes SBK14, SBK16, and SBK18 are only available up to 8".

STEP 3:	Insert Code for "B"
Locking	L
Cushion	C