


To determine proper liner sizing, two measurements are required:

Step #1: Take circumference measurement 2" (5-6cm) proximal to the distal end of the residual limb.

Step #2: Take length measurement from mid-patella tendon to the distal end of the residual limb.

Note: All Ultimate liners come with full fabric cover, in either locking or cushion (non-locking) versions.

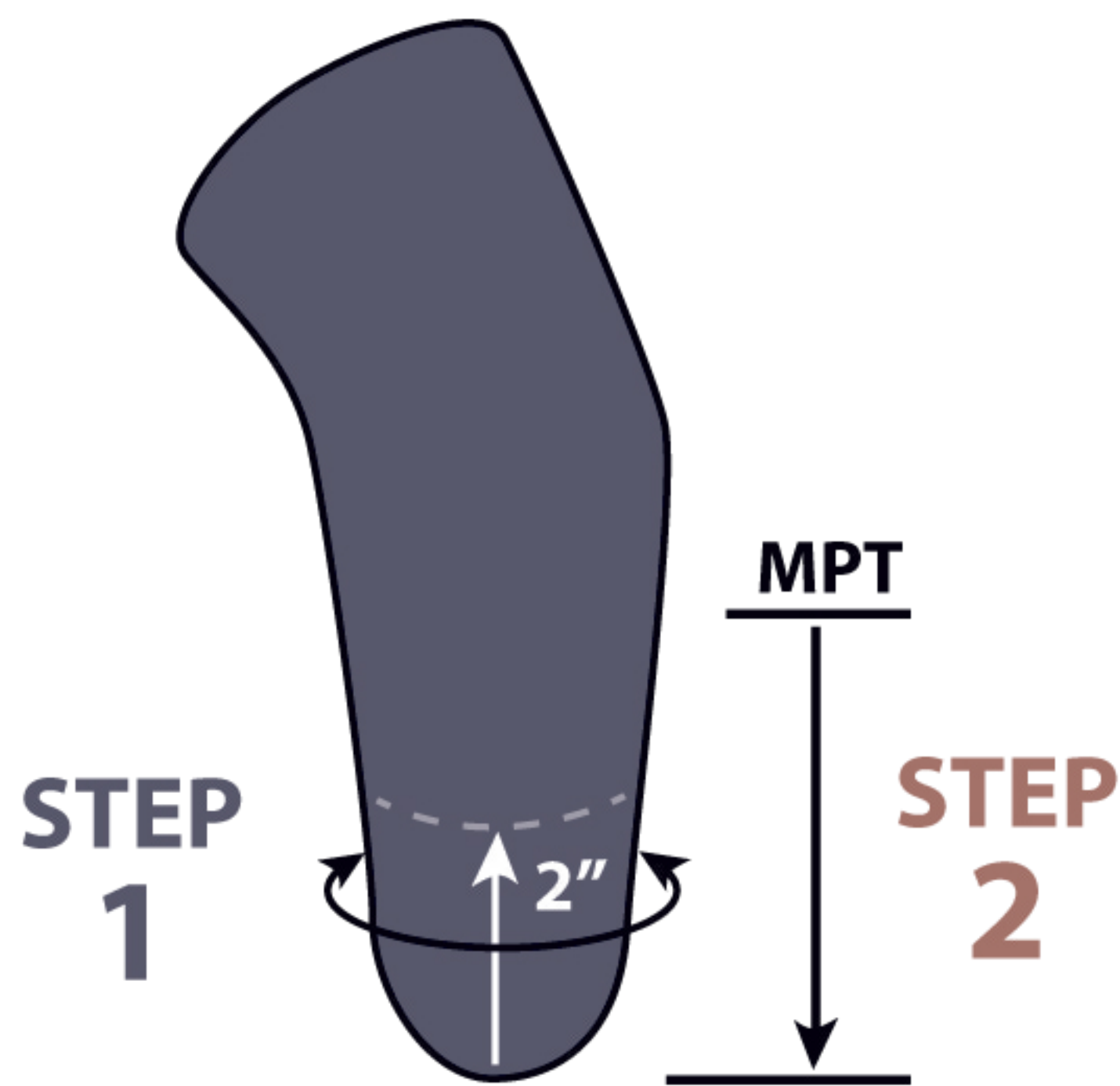

EXAMPLE: BK-U-L- 6 L

Limb Circ. Of 33" and an MPT to Distal Length of 6" plus Locking with Textile.

STEP 1: Limb Circ.	Choose Part Number
14 - 20 cm	BK-U-XS - A B
20 - 26 cm	BK-U-S - A B
26 - 32 cm	BK-U-M - A B
32 - 38 cm	BK-U-L - A B
38 - 42 cm	BK-U-XL - A B

STEP 2: MPT to Distal	Insert # for "A"
4"	4
6"	6
8"	8

STEP 3:	Insert Code for "B"
Locking	L
Cushion	C