English

PRODUCT LISTING

SKU	SIZE	SIDE
PA-WAHK-WC-SM	SMALL	UNIVERSAL
PA-WAHK-WC-MD	MEDIUM	UNIVERSAL
PA-WAHK-WC-LG	LARGE	UNIVERSAL
PA-WAHK-WC-XL	X-LARGE	UNIVERSAL
PA-WAHK-WC-2X	XX-LARGE	UNIVERSAL
PA-WAHK-WC-3X	3X-LARGE	UNIVERSAL
PA-WAHK-WC-4X	4X-LARGE	UNIVERSAL

Français

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N° DE RÉF.	TAILLE	CÔTÉ
PA-WAHK-WC-SM	PETIT	UNIVERSEL
PA-WAHK-WC-MD	MOYEN	UNIVERSEL
PA-WAHK-WC-LG	GRAND	UNIVERSEL
PA-WAHK-WC-XL	T-GRAND	UNIVERSEL
PA-WAHK-WC-2X	TT-GRAND	UNIVERSEL
PA-WAHK-WC-3X	3T-GRAND	UNIVERSEL
PA-WAHK-WC-4X	4T-GRAND	UNIVERSEL

PROACTIVE WEAR®

Wraparound Hinged Knee Genouillère enveloppante articulée

Instruction for Use
Mode d'emploi

PRODUCT DESCRIPTION

The ProActive Wear® Wraparound Hinged Knee is designed for ease of application and provides support, stabilization, and compression for ligament instabilities and injuries.

INDICATIONS FOR USE

- Ligament Sprains
- Osteoarthritis
- Patella Dislocations
- Knee Instability

INSTRUCTION FOR PROPER USE

1. Before applying, unfasten top and bottom straps.
2. Insert the foot into knee support and slide up and over the knee until the patella is centered in the patella cutout.
3. Feed the straps through the plastic rings, pull back and secure using the hook-and-loop closures. Straps should fit comfortably, without inhibiting circulation. Support should fit tight enough to provide even and constant support.

MAINTENANCE/CLEANING INSTRUCTIONS

Close all hook-and-loop closures. Hand wash using cold water and mild detergent. Rinse thoroughly. Allow product to air dry. Do not bleach.

SINGLE PATIENT USE

Single Patient Use Only

WARNINGS AND PRECAUTIONS

The ProActive Wear® Wraparound Hinged Knee should provide compression but should not be so tight that it restricts circulation. Consult your medical professional immediately if you experience any pain, swelling or unusual reactions to this product

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