



LYN VALVE BK PA0200 (US Threads) PA0200M (Metric Threads)



Instructions for use

AUTO EXPULSION & PUSH BUTTON FOR BELOW KNEE SUCTION SOCKET PROSTHESES

NOTE: Use pre-threaded housing (PA0222 or PA0222M) when installing BK Valve into thermoplastic materials. Drilling and tapping plastic materials may result in poor seal as plastics may stretch.

FEATURES

- Installation AFTER prosthetic construction is complete.
- Easy installation in sockets with wall thickness of 1/8" to 1/2" (Metric: 3.17mm-12.7mm) – no special tools required.
- Releases air reliably at 1/2 PSI.
- Indestructible low profile design - approximately 3/4" (Metric: 1.9mm) diameter and 1/2" high
- Disassembles for cleaning.
- Available in US and Metric thread

INSTALLATION

After prosthesis construction is complete, determine valve position and mark. Using a 5/16" (8.4mm-8.5mm) drill bit, drill square to the socket wall. Tap using a 3/8"-16 tap for non-metric (M10x1.5 tap for metric shank). Thread in valve and determine shank length required. Sand shank while installed using a Trautman router/coarse cone. Remove valve to buff shank. Apply silicone sealant around base of shank and "o" ring. Screw valve into socket—firmly hand tighten. Clean valve. (See below).

DISASSEMBLY AND CLEANING

1. Insert a screwdriver into the groove and twist gently to separate valve.
2. Remove spring.
3. Grip grooved stem and lift out diaphragm and vent spool assembly.
4. Use compressed air to remove any particulate that may have collected on the inner valve surfaces.
5. Be sure hole in shank is clear of debris.
6. Rinse with water if necessary.

REASSEMBLY

1. After cleaning, insert vent spool assembly into pocket.
2. Replace diaphragm with plastic side up.
3. Center spring and snap on cap.



Engineered Silicone Products, LLC
We're thinking on a higher level.

75 Mill Street, Suite #2
Newton NJ, 07860
ph. 888-932-7377 / 973-300-5120
fax 973-300-5125

www.wearesp.com



Medidas
Zaagmolen 40
3962 GB Wijk bij Duurstede
Netherlands
medidas@medidas.nl