

Aria™ Valve Tool



Please read all instructions carefully following all precautions and safety instructions

For use with Aria Valve (CA200-AV/AVM), Aria Valve Plus (CA200-AVP/AVPM), Aria Hybrid Valve (CA300-HYV), Aria Plate Valve (CA300-APV), Aria HV Plate Valve (CA300-AHVP), and Aria Auto Expulsion Valve Insert (CA400-V)

To Remove Caps on Aria Plate Valve, Aria HV Plate Valve and Aria Auto Expulsion Valve Insert:

Insert top prong of long metal end into space between valve cap and plate (for plate valves) and cap and body of Auto Expulsion Valve Insert. Gently twist tool to loosen cap . Note: Spring may pop out abruptly once pressure of cap is released.

To Remove Aria Valve, Aria Valve Plus and Aria Hybrid Valve From Socket Wall:

Once valve cap is removed by unscrewing it from valve, remove the spring, rubber diaphragm and center vent spool. This will expose 2 holes on inner face of valve body. Insert the 2 small metal prongs located on one end of valve tool into holes and use tool handle to unscrew valve from socket wall.



804 West Bartlett Rd.
Bartlett, IL 60103
info@cypressadaptive.com
www.cypressadaptive.com
888-715-8003
Fax: 888-715-4535



Medidas
Zaagmolen 40
3962 GB Wijk bij Duurstede
Netherlands
medidas@medidas.nl

Rev: 201809-AVT-ENG