

## Parts Included



Wave Suction Band



50ml Silicone  
Adhesive

### Wave Suction Band sizes

Part Number	Size	Circumference	Width	Thickness
CD130BM	Medium	23cm - 30cm	7.62cm	4mm
CD130BL	Large	30cm - 38cm	7.62cm	4mm
CD130BX	Extra Large	38cm - 48cm	7.62cm	4mm
CD130BXX	Extra Extra Large	48cm - 62cm	7.62cm	4mm

EN | Instructions for Use  
DE | Gebrauchsanweisung  
FR | Notice d'utilisation  
ES | Instrucciones para el uso  
IT | Istruzioni per l'uso  
NO | Bruksanvisning  
DA | Brugsanvisning  
SV | Bruksanvisning  
EL | Οδηγίες Χρήσης  
FI | Käyttöohjeet  
NL | Gebruiksaanwijzing  
PT | Instruções de Utilização

PL | Instrukcja użytkowania  
CS | Návod k použití  
TR | Kullanım Talimatları  
RU | Инструкция по использованию  
JA | 取扱説明書  
ZH | 中文说明书  
KO | 사용 설명서



[www.coyote.us/instructions-waveband](http://www.coyote.us/instructions-waveband)



Manufactured by Coyote®  
419 N. Curtis Rd., Boise, Idaho 83706  
(208) 429-0026 | [www.coyote.us](http://www.coyote.us)



# CD130BM

## Medium Wave Suction Band

Circumference 23cm - 30cm, Width 7cm, Thickness 5mm

### Fabrication Instructions



Weight limit: 265 lbs.

2-year warranty against manufacturer defects, excessive wear or breakage.

External Prosthetic Components



EC REP

Advena Limited  
Tower Business Centre  
2nd Flr, Tower Street  
Swatar, BKR 4013  
Malta

## Attaching Seal Band

Clean liner attachment area with acetone prior to gluing.





**1** With liner on patient position band in desired location.




**2** Mark band location on distal and proximal sides of band.




**3** For unconventional band position mark band in multiple locations.

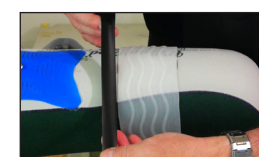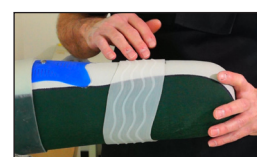

**5** Wrap the liner with electrical tape or vinyl tape next to the band without overlapping it.

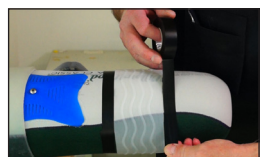

**6** Tape off both ends of the band.

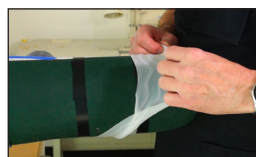


**7** Remove the band



**8** Add extra tape to protect a wider area.




**9** Put a bead of silicone adhesive around the liner about the thickness of a pencil.

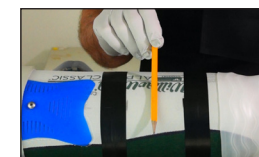


**11** Smooth the silicone out with a tongue depressor. Make sure to saturate the material.




**12** Check the edges for areas that need more silicone adhesive.

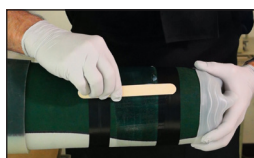


**13** Smooth the silicone out.

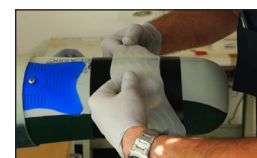


**14** Carefully place the seal band over the glued area.

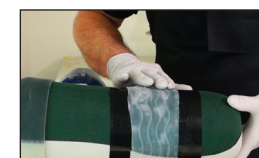


**15** Center the band with tape to the edges.

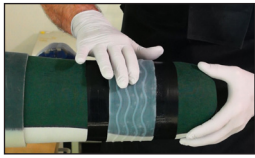


**16** Push the air bubbles out from under the band

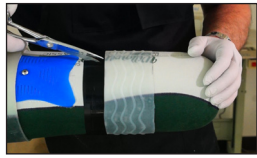


**17** Remove the tape from the liner.

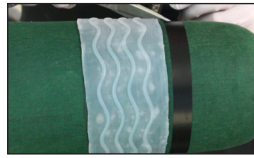

**18** Create a seal band edge with silicone adhesive.

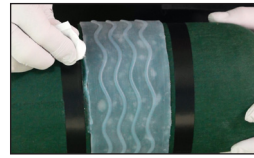


**19** Roll tape next to band edge approximately 1/4 inch space from band.

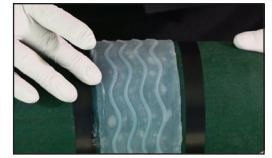

**20** Smooth silicone into fabric and band. Making a sealed edge.



**21** Remove tape.

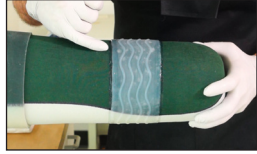

**22** Check your work and let the silicone set up for at least 8 hours before putting it on patient.

Wave Suction Band sizes				
Part Number	Size	Circumference	Width	Thickness
CD130BM	Medium	23cm - 30cm	7.62cm	4mm
CD130BL	Large	30cm - 38cm	7.62cm	4mm
CD130BX	Extra Large	38cm - 48cm	7.62cm	4mm
CD130BXX	Extra Extra Large	48cm - 62cm	7.62cm	4mm

Lay-up instructions are helpful hints on how to work with the lock and connector. Actual lay-ups are responsibility of the technician and/or practitioner.