BREATHABLE Coolfit Knee Brace

Part ID# CFB

Instructions of the Knee Brace

Part ID# CFB

The ALPS Knee Brace is constructed of Coolfit Breathable fabric with adjustable hinges. The brace is designed to prevent the severity of knee injury. The brace is available in pull-on and wrap-around styles.

There are a few of the built-in features from ALPS Knee Brace:

Hinges with flexion and extension adjustment,
Medial and lateral support,
Air slots behind knee,
Breathable fabric

Here are some clinical indications for using Prophylactic Knee Braces:

MCL protection against valgus knee stresses,
Protection after re-injuring previous MCL injury,
Athletes at high risk for MCL injury,
Rotational control is limited in ACL-deficient knees.

Sizing

Measure the circumference 10 cm (4 in.) above the kneecap

Select a size inferior to the measured circumference

Example 1: Measure circumference 11 inches

Select size XS

Example 2: Measure circumference 14 inches

Select size S

Size	Fits (Inches)
X-Small	11" - 13"
Small	13.5" - 15"
Medium	15.5" - 17"
Large	17.5" - 22"
X-Large	22.5" - 24"
XX-Large	24.5" - 27"
XXX-Large	27.5" - 30"

Knee Brace Hinged Pull-on Application Instructions

1. Before your application, adjust hinges' angle according to individual requirements, and then insert them into hinge cover. Always loosen straps applying brace.
2. Start from a sitting position when applying the knee brace. Pull the entire brace past the heel, then stand up and pull knee brace up centering the knee brace over the patella by using both hands.

Knee Brace Hinged Wrap-around Application Instructions

1. Before your application, adjust hinges' angle according to individual requirements, and then insert them into hinge cover. Always loosen straps applying brace
2. Start from a sitting position with the knee bent slightly
3. Keeping hole centered on patella. Keep air slots positioned behind knee.

Note: Before you apply the knee brace, please keep your skin dry.

Hinge Set Up Instructions:

1. Remove hinges out of the brace pockets.
2. Remove metal hinges out of cover while using caution as hinge stop is not secured.

3. Hinges have an engraved “R” for right side and “L” for left side. Place hinge stop to appropriate side and angle degree. Select degree 0 for full leg extension and 15, 30 or 45° to reduce the degree extension of the knee.
4. Once STOP hinges are in place replace fabric covers and relocate back into brace pockets.

NOTE: Use pull straps located on each side of brace to secure brace on leg. (ALPS Logo is indicating front of brace)

Cleaning Instructions

Hand wash in cool water. Do not bleach, twist or wring. Reshape and dry flat.

Warranty

30-days satisfaction guaranteed, no questions asked

3 month warranty against manufactures defects for one

To obtain an Alps Return Authorization Number call Alps Customer service between the hours of 8am and 5pm eastern time.

CE Marked

EC	REP
----	-----

CEpartner4U

3951db: 13. NL

tel: +31 (0)6.516.536.26

CAN# 2637 DOC# 900086 (A)

Alps South Corporation
2895 42nd Avenue North
St. Petersburg, FL. 33714
USA

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Alps South Corporation – Filiale Italiana