



# **PDS-PadSil**

Silicone Distal Pads



**Alps PadSil** is made from the non-allergenic properties of silicone. Alps Padsils provide more cushioning, softness and durability for your patient due to the unique compression set characteristics of silicone, these pads will not take a set even with prolonged hard use. Padsils can be used in practically every type of socket, even inside silicone suction sockets and liners. Padsils are machinable and can be shaped to fit with standard tools. ALPS PadSil (**PDS-xx**) is **available in 5 sizes S, M, L, XL, XXL**.

### Sizing Instructions

#### Step 1:

Measure the circumference in cm at the beginning of the cylindrical section approximately 6 cm from the bottom of the distal end. This is the measured size.

#### Step 2:

Find the measured Size from Step 1 in the following chart. Read across to the find the recommended size.

Limb Circumference (cm)	PadSil
14-19 cm	S
20-23 cm	M
24-27 cm	L
28-31 cm	XL
32-35 cm	XXL

### Alps Satisfaction Guarantee

Alps offers a no hassle return policy within **30 days** from date of purchase. If you are unsure of the proper size of PadSil to order, you can order two sizes from Alps Customer Service. Once received, decide which size fits best. You can then call Alps Customer Service for a Return Authorization Number and return the improper size for full credit of the product cost within 30 days from date of purchase (excludes freight charges).

## Warranty

Alps PadSils are warranted against manufacture's defects for **6 months** from the date of purchase. Alps South Corp. warrants only that its products will meet its specifications. **THERE IS NO WARRANTY OF MERCHANTABILITY OR FITNESS FOR USE, NOR ANY OTHER EXPESS OR IMPLIED WARRANTIES.** The user's exclusive remedy and Alps South Corp.'s sole liability is limited to the replacement of any product shown to be otherwise than as warranted. Alps South Corp. will not be liable for incidental or consequential damages of any kind.

To obtain an Alps Return Authorization Number (RAN#) call Alps Customer service between the hours of 8 am and 5 pm EST and provide Alps representative with the following information.

- 1. Customer ID Number**
- 2. Invoice Number**
- 3. Date of Invoice**
- 4. Nature of the return**

The RAN# must be displayed on the exterior of the returned item box or it will be refused at the dock.

Any products returned to Alps after 30 days without manufacturing defects will be subject to a 25% restocking charge.

900027 Rev E, CAN#1195  
PDS

CE Marked

EC	REP
----	-----

CEpartner4U

3951db: 13. NL

tel: +31 (0)6.516.536.26



**Alps**  
**2895 42<sup>nd</sup> Ave. N.**  
**St. Petersburg, FL. 33714**  
**USA**  
**Tel: 1-727-528-8566**  
**Fax: 1-727-528-8862**  
[Info@easyliner.com](mailto:Info@easyliner.com)  
[www.easyliner.com](http://www.easyliner.com)