



# **SFR - Fabric Reinforced EasySleeve**



Covered by one or more of the following patents

6,552,109B1

6,867,253B1

7,344,568B2

7,709,570B2

## **ALPS Fabric Reinforced EasySleeve SFR Suspension Sleeve**

### **Instructions**

The ALPS EasySleeve - High Performance Easysleeve (SFR) is made with a very comfortable soft gel material. The soft feeling of the gel reduces the occurrence of rashes and skin breakdowns. The SFR with or without an expulsion valve can be used with the ELFR which is used with any type of proximal suspension including supracondylar and straps.

### **Sizing**

Measure the stump circumference 10cm above patella of the residual limb.

Select a size inferior to the measured circumference.

Example 1: Measured circumference 24

Select size 24

Example 2: Measured circumference 27

Select size 26

### **Measurements taken 10cm above patella.**

<b>Measured size (cm)</b>	<b>3mm</b>	<b>6mm</b>
16 – 19 cm	SFR 10-3	SFR 10-6
20 - 24 cm	SFR 16-3	SFR 16-6
25 - 30 cm	SFR 20-3	SFR 20-6
31 - 34 cm	SFR 24-3	SFR 24-6
35 - 37cm	SFR 26-3	SFR 26-6
38 - 40 cm	SFR 28-3	SFR 28-6
41 - 45 cm	SFR 32-3	SFR 32-6
46 - 60 cm	SFR 38-3	SFR 38-6
61 - 70 cm	SFR 44-3	SFR 44-6

### **Donning the EasySleeve** -SFR

- 1). Although it is not necessary to apply any type of lubricant to the residual limb, you may use either talcum powder or silicone skin lotion -- Alps SL108. **Avoid petroleum jelly and other oil based lubricants**, as they will damage the sleeve.
- 2). Apply talcum powder to the inside of the EasySleeve™.
- 3). Invert the EasySleeve so that the inside is facing out and roll the sleeve up onto the socket.
- 4). Introduce the residual limb into the socket.
- 5). Roll the EasySleeve onto the proximal end of the residual limb.

### **Cleaning the EasySleeve**

**Daily:** Wash the sleeve and the residual limb with warm water and a neutral soap such as Alps Cleanser PD 595 or other hypoallergenic pH-balanced soap. Use a wash cloth, but do not rub, nor scrub on the gel or you will damage the EasySleeve. Rinse Well. Wipe the sleeve dry with a clean, dry towel and apply a light dusting of talcum powder to the outside of the sleeve.

**Weekly:** Place a small amount of rubbing alcohol on a soft, clean cloth. Wipe the sleeve with the cloth thoroughly, wipe dry with a towel and apply a light dusting of talcum powder to the outside of the sleeve. **DO NOT** soak the sleeve in alcohol.

To remove a stain on the fabric side, use a cloth dampened in nail polish remover (acetone) and rub until clean **DO NOT SOAK IN ACETONE OR YOU WILL DISSOLVE THE GEL.**

## Warranty

EasySleeve suspension sleeves are covered by our 30-day satisfaction guarantee, no questions asked. Full credit will be applied against the purchase price (excluding freight charges). Furthermore, they are guaranteed against manufacturer's defects for 90 days.

To obtain an Alps Return Authorization Number (RAN) call Alps Customer Service between the hours of 8 am and 5 p.m. EST and provide an Alps representative with the following information: **Customer ID # -- Invoice # -- Date of Invoice – Nature of the Return**

The RAN must be displayed on the exterior of the returned item box or it will be refused at the dock.

**Warning:** Specific exclusions from this warranty are: cuts, holes, rips due to hits, scrapes, etc., and abrasions due to improper taping of the socket.

CE Marked

EC	REP
----	-----

CEpartner4U

3951db: 13. NL

tel: +31 (0)6.516.536.26

900036 Rev K, CAN#2946  
SFR

Alps South LLC  
2895 42<sup>nd</sup> Avenue North  
St. Petersburg, FL. 33714  
USA  
Tel: 1-727-528-8566  
Fax: 1-727-528-8862  
[Info@easyliner.com](mailto:Info@easyliner.com)  
[www.easyliner.com](http://www.easyliner.com)