

You Customize the Length!

The Neo-Flex is a cost-effective neoprene suspension sleeve that will hold a seal for passive suction when used with a valve in addition to providing a secure mechanical suspension of the prosthesis. It is constructed of a durable latex -free neoprene and has a flexed shape for comfort and free range of motion. The proximal and distal length of the Neo-Flex Suspension Sleeve may be customized for your patient by trimming at any point in between the trim tacks located along the posterior seam of sleeve.

Trimming Instructions:

If length adjustment of proximal and/or distal end of sleeve is desired, choose any spot BETWEEN the reinforcement tacks located along the posterior seam of the sleeve and mark cutting line with chalk or marker. Place sleeve on its side and cut from posterior to anterior along your cutting line using sharp scissors.





For use as a prosthetic suspension aid.

Warning:

Do Not use over broken or irritated skin. Discontinue use of sleeve if any skin irritation, blisters or rash develop with use.

Do not wear while sleeping or use if you have circulatory problems.

Care Instructions:

Wash sleeve with mild detergent and cool water and lay flat to dry. Do not use excessive heat, chlorine bleach, or solvents.

Warranty:

Defects in manufacturing or materials used will be covered for 90 days from the date of purchase. Defective item must be returned clean and dry and will either be replaced or repaired.

Neo-Flex[™] Suspension Sleeve Sizing

For proper sizing, take thigh circumference measurement 5" proximal to top edge of the patella and refer to sizing chart.

Neo-Flex is available in:
Golden TanCharcoal Gray

Limb Measurement	Sleeve Size	Ref no. Golden Tan	Ref no. Charcoal Gray
10.5"-12.5"	0	CA700-FP-T0	CA700-FP-G0
11.5"-13"	1	CA700-FP-T1	CA700-FP-G1
12.5"-14"	1.5	CA700-FP-T15	CA700-FP-G15
14"-16"	2	CA700-FP-T2	CA700-FP-G2
15"-17"	2.5	CA700-FP-T25	CA700-FP-G25
16"-18"	3	CA700-FP-T3	CA700-FP-G3
18.5"-20	4	CA700-FP-T4	CA700-FP-G4
21"-23"	5	CA700-FP-T5	CA700-FP-G5