

Ortho Innovations' Pre-fabricated Products Measurement Information & Size Charts

Pre-fabricated Knee Extension Brace

How to measure for a Pre-fabricated Knee Brace.

- Position leg in maximum extension.
- A circumference measurement should be taken around the 2 leg at each level.
 - Level A is located 9.5 inches (25 cm) above joint center.
 - Level B is located at the joint center.
 - Level C is located 9.5 inches (25 cm) below joint center.

Measurement for Pre-fabricated Knee Extension Brace			
	Circumference		
		Inches	Centimeters
Small	Α	12.75 - 16.5	32.4 - 41.9
	В	11.75 - 14.5	29.8 - 36.9
	С	8 - 11.75	20.3 - 29.8
Medium	Α	18 - 21	45.7 - 53.3
	В	15 - 17	38.1 - 43.2
	С	9.5 - 12.5	24.1 - 31.8
Large	A	21 - 24	53.3 - 61
	В	17 - 20	43.2 - 50.8
	С	12.75 - 16.5	32.4 - 41.9

